Horry County Fire Rescue
Office of the
Assistant Chiefs

Battalion One: Douglas Cline
Office Phone: 843.365.2384
Battalion Two: Justin L. Gibbins
Office Phone: 843.903.4094
Battalion Three: P. Scott Thompson
Office Phone: 843.365.1885

Operational Directive 2011-01

To: All Personnel

Date: October 19, 2011

Re: Lancets


All personnel need to immediately check their apparatus and station inventory for the old style lancets. The old lancets are non-compliant with safety regulations and need to be discarded immediately. The compliant lancet carried by the Department is a SurgiLance Safety Lancet(see attached.) All other lancets should be discarded immediately.

Horry County Fire Rescue
Office of the
Assistant Chiefs

Battalion One: Douglas Cline
Office Phone: 843.365.2384
Battalion Two: Justin L. Gibbins
Office Phone: 843.903.4094
Battalion Three: P. Scott Thompson
Office Phone: 843.365.1885

USER GUIDE / GUÍA DEL USUARIO / MODE D'UTILISATION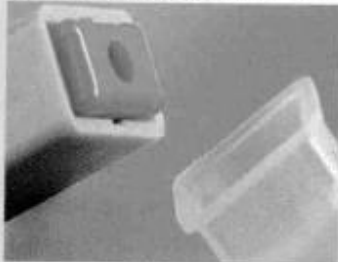

Simply clean test site with isopropyl alcohol and wipe dry. Remove protective cap from the safety lancet.

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Place red raised platform end of safety lancet on the test site.

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Gently push the safety lancet down against test site to activate.

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Once used, simply discard lancet into a refuse container.

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